

Multi-Shipper with Cold Pack



Kentucky Public Health
Prevent. Promote. Protect.

Supplies Needed for Collection and Packaging



Blood tube or other specimen collection container



Multishipper with Cold Pack

Contents: Tube Shuttle, 95kPa Bag with absorbent, Cold Packs, Styrofoam Cooler and Outer Cardboard Box.



Outreach or Lab Form

Collection and Labeling of Specimen

- 1) Collect patient specimen in accordance with package insert .
- 2) Make sure two identifiers are on the specimen vial.
- 3) Fill out lab or Outreach form completely.

Packaging of Specimen

- 1) Place specimen tube(s) into tube shuttle.
- 2) Roll up tube shuttle/specimens and place in 95kPa bag.
If shipping single container place in 95kbag with absorbent.
- 3) Place sample bag on top of frozen freezer blocks and replace styrofoam lid.
- 4) Put Outreach form on top of styrofoam lid and close outer box.
Place laboratory label on the top of the box.

