

# Multi-Shipper



Kentucky Public Health  
Prevent. Promote. Protect.

## Supplies Needed for Collection and Packaging



Blood tube or other specimen collection container



Multishipper

Contents: Tube Shuttle, 95kPa medium canister, bubble wrap sheet and Outer Cardboard Box. with correct markings.



Outreach or Lab Form

## Collection and Labeling of Specimen

- 1) Collect patient specimen in accordance with package insert .
- 2) Make sure two identifiers are on the specimen container.
- 3) Fill out lab or Outreach form completely.

## Packaging of Specimen

- 1) Place specimen tube(s) into tube shuttle.
- 2) Roll up tube shuttle/specimens and place in 95kPa medium canister.
- 3) Place bubble wrap and lab form around outside of 95kPa medium canister.
- 4) Put medium canister in box. Place laboratory label on the top of the box.



- Note: 1. Do not put biohazard label on outside of box. Biohazard label should be on the 95kPa medium canister.  
2. A small piece of tape can be used to seal outer box.