

# Multi-Shipper for Ambient Specimens

## Supplies Needed for Collection and Packaging



Blood tube or other  
specimen collection container



Multishipper

Contents: Tube Shuttle,  
95kPa medium canister,  
bubble wrap sheet and  
Outer Cardboard Box. with  
correct markings.



Outreach Form

## Collection and Labeling of Specimen

- 1) Collect patient specimen in accordance with package insert .
- 2) Make sure two identifiers are on the specimen container.
- 3) Fill out lab or Outreach form completely.

## Packaging of Specimen

- 1) Place specimen tube(s) into tube shuttle.
  - 2) Roll up tube shuttle/specimens and place in 95kPa medium canister.
  - 3) Place bubble wrap and lab form around outside of 95kPa medium canister.
  - 4) Put medium canister in box. Place laboratory label on the top of the box.
- 
1. Do not put biohazard label on outside of box. Biohazard label should be on the 95kPa medium canister.
  2. A small piece of tape can be used to seal outer box.



When shipping via FedEx place box inside UN3373 Pak



Refer to 49CFR 173.199 for current regulations on packaging and shipping of Category B infectious substances